COLLECTION CHAUSSETTES · SOCKS COLLECTION

FeetPAD ORLIMAN

Français

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English

INSTRUCTIONS FOR USE AND PRESERVATION
Dear customer, Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING
Washing instructions:
- Wash daily.
- The socks are machine washable using the delicates program at 30°C / Or wash by hand.
- Only use detergent for delicate clothes. Do not use fabric softener because it can close the pores and matt the fibres together.
- Never dry the socks in the sun or on a radiator. Dry horizontally in the shade or hang them out.
- Do not use chemical cleaning agents or bleach and do not iron.

SUITABLE FOR
Compression socks - Travel
- Improved venous return, thus making the sock suitable for all users who make journeys in which they have to remain seated for many hours.

Socks for comfort and protection - Diabetic feet
- Diabetic foot.
- Candidiasis.
- Mycosis.
- Eczema.
- Preventing the proliferation of bacteria and fungi (bacteriostatic).

FITTING INSTRUCTIONS
Compression socks - Travel
- Before putting on the sock, make sure your foot is completely dry.
- Turn the sock inside out as far as the heel.
- Insert your foot as far as the heel.
- Pull the sock up over the lower part of your leg.
- Make sure the fabric is evenly distributed.
- Do not fold over the top of the sock.

PRECAUTIONS - WARNINGS
The constructive material is inflammable. Do not expose these products to situations where they could ignite. If this were the case remove them quickly and use the necessary measures to put it out. The material used is hypoallergenic, however we cannot guarantee 100% that in some determined cases it will not produce allergies, if this occurs, remove the product and see the doctor that prescribed it. For small inconvenience caused by sweat, we recommend using an interface to separate the skin from contact with the tissue. If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Contraindicated in open scars with swelling, redness and accumulation heat.

MANUFACTURE - CHARACTERISTICS
The material features used in production have been tested and approved and meet

all the European quality requirements. All the products are produced with high quality materials and offer an unbeatable comfort and quality of use. All products offer restraint, stability and compression for the optimum treatment of the pathologies for which they have been designed.

RECOMMENDATIONS FOR PRESERVATION AND CLEANING
Washing instructions:
- Wash daily.
- The socks are machine washable using the delicates program at 30°C / Or wash by hand.
- Only use detergent for delicate clothes. Do not use fabric softener because it can close the pores and matt the fibres together.
- Never dry the socks in the sun or on a radiator. Dry horizontally in the shade or hang them out.
- Do not use chemical cleaning agents or bleach and do not iron.

GUARANTEES
ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed. ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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