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**INDICATIONS**

Absorbs impacts thanks to its thin polymer gel pad. Prevents the formation of corns. Stays in place and provides comfort thanks to its thin band of elastic fabric.

**FITTING INSTRUCTION**

Insert the foot into the cushion, place the retention tongue between the big toe and the index toe, next place the socks, tights or panty hose over it. Carefully check the size as excessive compression can cause vascular type.

**WARNING**

Do not place product over an open wound. If irritation, discomfort, or bad circulation occurs stop usage. Remove the product for at 3 or 4 hours a day to allow the skin breathe. Carefully check the size as excessive compression can cause vascular type interruptions. Do not place near sources of heat. KEEP OUT OF REACH OF CHILDREN.

**COMPOSITION**

Polymer Gel.  
Lined: Polyamide and Elastomer.

**RECOMMENDED CARE**

It is advisable to wash the product by hand (do not rub). Leave to air dry. Do not iron.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE-EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthesis.