Lilaë

protect

soft

FR - TALONNETTES DE PROTECTION & SEMELLES DE CONFORT

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INDICATIONS

Lilaë Protect® silicone Heel cup:

Relieves pain associated with talalgia, bursitis and Achilles tendinitis. Relieves joint pain. Decreases the pressure on the heel.

Lilaë Soft® comfort insoles:

Relieves pain associated with metatarsalgia, talalgia and tendinopathy. Reduces the pressure on the support points. Relieves aching or tired feet Limits the sagging of the arch.

COMPOSITION

Lilaë Protect® silicone Heel cup:

100% silicone.

Lilaë Soft® comfort insoles:

Insole covering:

Latex with polyester fabric and silver thread.

Insoles:

100% polyurethane.

Reinforcement:

Thermoplastic with polyester fabric.

Pads: Latex.

SAFETY INSTRUCTIONS

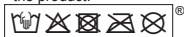
This product must be fitted by a health professional (pharmacist, orthopaedist, physiotherapist, etc.). If rash, irritation or swelling to remove the product and consult a doctor or prosthetist. Protect any part of the body to which an analgesic may have been applied before fitting the product. It is not recommended for other patients to reuse this product. In the event of a serious incident with the device, inform the manufacturer and the competent authority.

Do not expose these products to situations where they could ignite.

Products marked with the latex symbol contain natural rubber latex and can cause allergic reactions in latex-sensitive individuals.

WASHING INSTRUCTIONS

- Wash by hand with neutral soap (max. 30 °C),
- Rinse carefully,
- Allow to dry in the open air,
- Do not dry in a dryer,
- Do not use bleach,
- Do not hang up or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sun light etc. During use or during washing do not use alcohols, ointments or dissolvent liquids. If the orthosis is not properly dried any detergent residues could irritate the skin and deteriorate the product.

Store at room temperature.

In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

FITTING

To achieve the highest degree of therapeutic effectiveness for the various conditions and to prolong the service life of the product, it is important to choose the right size (the packaging contains a size guide).

- 1- Choose the right size.
- 2- Use a heel cup or insole for each foot, unless the doctor indicates otherwise.
- 3- Insert the heel cup or insole into your shoe with the flat part facing downwards and then place your foot into the anatomically shaped part.
- 4- You are recommended to wear loose-fitting shoes and socks.
- 5- It may take 2 or 3 days to get used to the insole and for it to feel comfortable.

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