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INDICATIONS

Enables the joint to be aligned thanks to its toe separator. Relieves by cushioning pressure of footwear thanks to its moisturizing and soothing polymer gel.

POLYMER GEL TECHNOLOGY Thanks to its composition of medical mineral oil, moisturize, soften, protect, and cushion the affected area.

FITTING INSTRUCTION

Place the shield over the big toe. Position the gel pad over the joint with the bunion and the separator between the toes. Note: if you wear dress shoes it is recommended to spray inside the shoe with a waterproof product.

WARNING

Do not place product over an open wound. If irritation, discomfort, or bad circulation occurs stop usage. Remove the product for at 3 or 4 hours a day to allow the skin breathe. Carefully check the size as excessive compression can cause vascular type interruptions. Do not place near sources of heat.

KEEP OUT OF REACH OF CHILDREN.

COMPOSITION

Polymer Gel.

RECOMMENDED CARE

It is advisable to wash the product by hand (do not rub). Leave to air dry.

**REGULATION**

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE-EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthesis.