# Compresse de *froid*

Bolsa fría  
Cold pack

ORLIMAN®

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This product is class I medical device. The manufacturing date is indicated next to the symbol **LOT**, in lack thereof, included in the batch number which can be found on the packaging label as **LOT**, in the following way: the second and third digits represent the year of manufacture and the fourth and fifth digits represent the month. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.

**PRECAUTIONS :** It's not recommended to use this product in case of allergy or on irritated skin. If you experience any pain or swelling or notice any change in the way it feels or any unusual symptoms, seek medical advice immediately. Do not wear the device next to the skin. Cover any part of the body where an analgesic may have been applied before applying the device. It is strongly advised that the device is not used for any other patient. Do not use for any other purpose. Do not open or drill. Do not use for infants. Do not use as a teething ring. Do not swallow content. Always use the product under adult supervision. In case of contact with eyes or skin, rinse immediately with clear water during 15 min. Seek medical advice. If swallowed, do not induce vomiting and immediately seek medical advice.

**INDICATIONS :** Reusable cold compress, rapidly relieves, bruises, sprains, strains, Tendinitis.

**COMPOSITION :** Gel : Sodium 38 %, acrylic acid (gelling agent) 25 %, polyacrylamide (flocculant) 25 %, hydroxide 10 %, water 2 %. **Bag :** PE 100 %.

**CLEANING ADVICE :** Handwash in soapy water (max. 30°C). Rinse thoroughly. Allow to dry naturally. Do not tumble dry. Do not bleach. Do not use detergent. Store at room temperature.